Spots (616S134), Sticky Spots (616S132)

EN Instructions for use	2
DE Gebrauchsanweisung	4
FR Instructions d'utilisation	7
IT Istruzioni per l'uso	10
ES Instrucciones de uso	12
PT Manual de utilização	15
NL Gebruiksaanwijzing	18
SV Bruksanvisning	21
EL Οδηγίες χρήσης	24
JA 取扱説明書	27
ZH 使用说明书	29

INFORMATION

Last update: 2020-04-20

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document in a safe place.

Explanation of symbols

English

 **CAUTION** Warnings regarding possible risks of accident or injury.

 **INFORMATION** Additional information on the fitting / use.

1 Description

1.1 *Indications for use*

The Spots 616S134 have been designed as volume management tool for prosthetic sockets. The Sticky Spots 616S132 have been designed for weight bearing and/or pressure relief in prosthetic sockets or orthoses (e.g. AFO, TLSO). Both types of Spots are an ideal tool where the skin is vulnerable to pressure.

1.2 *Safety instructions*

 **CAUTION**

Skin or tissue damages due to direct use of Spots or Sticky Spots on the skin. Uncovered PUR material is very sticky. Always apply Spots with the sticky side to the socket, liner or orthosis. Never use directly on the skin!

1.3 Function

Both types of Spots are made of polyurethane (PUR) which has unique flow and recover characteristics. Thereby they provide pressure relief or can fill distal gaps in prosthetic sockets (if used for volume management).

2 Handling

2.1 Spots 616S134

Place the Spot in distal portion of socket. If you are using a socket with an air expulsion valve in that area, a sheath or sock must be on the outside of the Spot to allow air to flow through the valve.

2.2 Sticky Spots 616S132

Sticky Spots have an adhesive on one side and include an oversized adhesive Lycra® cover. Simply clean the area for the Spot, remove backing, and stick it in place. Cover with Lycra® cover to keep it in place. Can be used in plastic applications and laminations.

INFORMATION

If used as fixed placement with sockets or orthoses proceed as follows:

- a) Attach the spot at desired location on positive plaster model.
- b) Pull plastic or laminate over it.
- c) Replace with Sticky Spot for final fitting.

3 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

3.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

3.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

3.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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