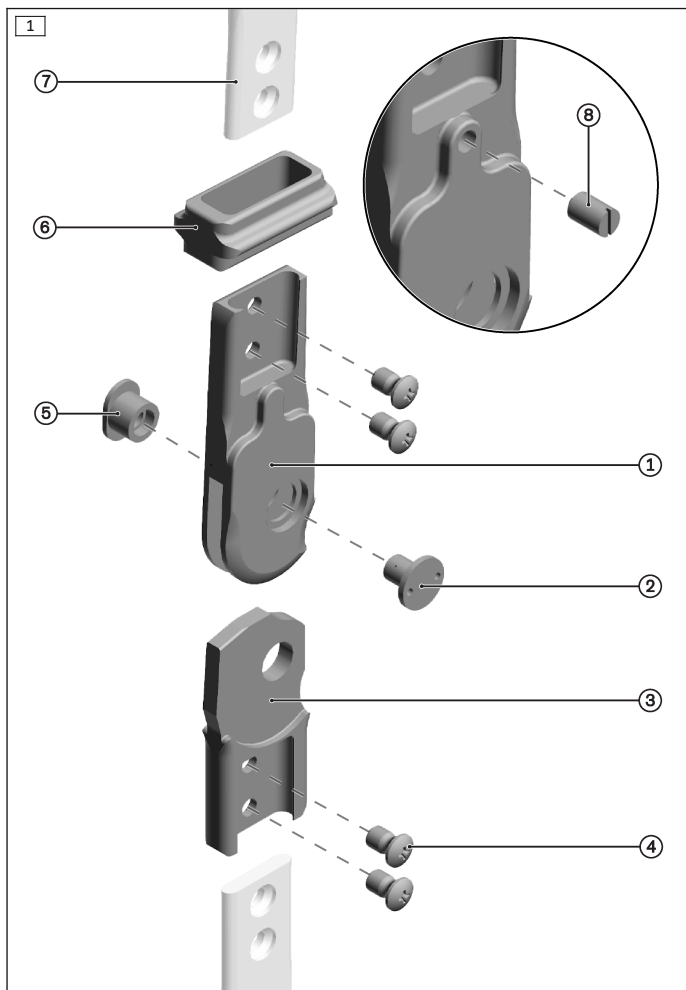
17B20*, 17B21*, 17B42*

DE Gebrauchsanweisung	3
EN Instructions for use	9
FR Instructions d'utilisation	14
IT Istruzioni per l'uso	21
ES Instrucciones de uso	27
PT Manual de utilização	33
NL Gebruiksaanwijzing	40
SV Bruksanvisning	46
DA Brugsanvisning	51
NO Bruksanvisning	57
PL Instrukcja użytkowania	63
HU Használati utasítás	69
CS Návod k použití	75
TR Kullanma talimatı	80
EL Οδηγίες χρήσης	86
RU Руководство по применению	93

INFORMATION

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INFORMATION

Date of last update: 2021-06-07

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the processing of the 17B20*, 17B21* and 17B42* orthotic knee joints.

2 Product description

2.1 Available sizes

Article number	Medial and lateral joint	System width	Material
17B20=L/R16	Outside straight, inside contoured	16 mm	Stainless steel
17B20=L/R20		20 mm	
17B21=16	Outside and inside straight	16 mm	
17B21=20		20 mm	
17B42=L/R16	Outside straight, inside bottom contoured	16 mm	
17B42=L/R20		20 mm	

2.2 Components/design

Scope of delivery (see fig. 1)			
Item	Quantity [Piece (s)]	Designation	Article number
1	1	Upper joint section	
2	1	Joint screw	501S34=M6
3	4	Lower joint section	
4	1	Screws	501T7*
5	1	Bearing nut	17Y93*
6	1	Ring lock	17Y31=25x9.5

Not included in scope of delivery			
Item	Quantity [Piece (s)]	Designation	Article number
(see fig. 1), item 7	1	Side bar	17B38*, 17B39*, 17B40*, 17B41*, 17B84*
not illustrated	1	Lamination bar	17Y128*, 17Y129*
Item 8	1	Spring-loaded thrust piece	501D1

3 Intended use

3.1 Indications for use

The orthotic joint is intended **exclusively** for use as a pair for orthotic fittings of the lower limbs on one patient.

3.2 Indications

- Partial or total paralysis of the leg muscles
- Orthopaedic diseases of the lower limbs

Indications must be determined by the physician.

3.3 Lifetime

The product is designed for a lifetime of **3 years** when used as intended and assembled professionally.

3.4 Qualification

Patients may be fitted with the product only by trained qualified personnel. The qualified personnel must be familiar with the handling of the various techniques, materials, machines and tools.

3.5 Combination possibilities

The 17B20*, 17B21* and 17B42* orthotic knee joints can be combined with the 501D1 spring-loaded thrust piece.

4 Safety

4.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

4.2 Safety instructions

 **CAUTION**

Incorrect alignment or assembly

Injuries due to changes in or loss of functionality

- ▶ The product may only be installed by trained, qualified personnel.
- ▶ Observe the alignment and assembly instructions.

 **CAUTION**

Excessive strain on load-bearing components

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.
- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

 **CAUTION**

Mechanical damage to the product

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

⚠ CAUTION

Excessive strain due to use on more than one patient

Risk of injury and loss of functionality as well as damage to the product

- ▶ Use the product on only one patient.
- ▶ Observe the maintenance recommendations.

⚠ CAUTION

Improper use of the lock

Risk of injury due to unexpected deactivation of the orthotic joint lock

- ▶ Fully extend the orthotic joint to activate the lock. Check whether the lock is engaged.
- ▶ Only deactivate the lock when you are able to control the flexion of the orthotic joint.

⚠ CAUTION

Clothing getting caught in the orthotic joint

Risk of injury due to loss of orthotic joint functionality

- ▶ Make sure your clothing does not get caught in the orthotic joint.

NOTICE

Thermal overloading of the product

Damage due to improper thermal treatment

- ▶ Do not carry out any heat treatment at temperatures above **300 °C (570 °F)**.
- ▶ Prior to thermal treatment, remove all temperature-critical components (such as plastic parts).

NOTICE

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

INFORMATION

Increased wear and tear

The orthotic knee joints are intended only for use in combination with free-motion or foot lifting orthotic ankle joints. Wear occurs more rapidly in combinations involving orthotic ankle joints with a dorsal stop.

5 Preparing the product for use

NOTICE

Orthotic joints not aligned in parallel

Malfunction of the locking mechanism

- ▶ Use the 743R6 orthotic joint alignment fixture.
- ▶ Make sure the orthotic joints lock simultaneously in the extension stop.

INFORMATION

Corrosion protection

The side bars and orthotic joints are resistant to corrosion. Fine grinding and polishing increase corrosion protection. Use sintering powder (e. g. 618T40) for coating.

Gluing in bars

Required materials: 636W28* special adhesive, degreasing cleaner

The bars must be glued in the insertion zones to ensure stability.

- 1) Clean the connecting surfaces with a degreasing cleaner.
- 2) Apply the special adhesive to the connecting surfaces.
- 3) Insert the bars.
- 4) Secure with the enclosed screws.
- 5) Allow to cure for at least **4 hours**.

INFORMATION: Final bonding strength will be reached after 16 hours.

6 Cleaning

After contact with water containing salt, chlorine or soap, or if they get dirty, the orthotic joints must be promptly cleaned.

- 1) Rinse the orthotic joints with clean fresh water.
- 2) Dry the orthotic joints with a cloth or allow them to air dry. Avoid exposure to direct heat (e.g. from an oven or radiator).

7 Maintenance

CAUTION

Wear and tear of the locking mechanism

Injuries due to changes in or loss of functionality

- ▶ In case of wear and tear of the locking mechanism, replace the lock wedge or the lock lever and/or the lower joint section.

INFORMATION

The product may be exposed to increased loads by the patient.

- ▶ Shorten the maintenance intervals according to the expected loads.

The manufacturer requires at least a semi-annual inspection of the product to verify functionality and check for wear.

Only use 633F7 special lubricant.

Spare parts are listed in the section "Components/design".

8 Disposal

Dispose of the product in accordance with national regulations.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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