Sealing sleeves

DE Gebrauchsanweisung	2	CS Návod k použití.....	17
EN Instructions for use	3	RO Instrucțiuni de utilizare.....	18
FR Instructions d'utilisation.....	4	HR Upute za uporabu	19
IT Istruzioni per l'uso	5	SL Navodila za uporabo	21
ES Instrucciones de uso	6	SK Návod na používanie	22
PT Manual de utilização.....	8	BG Инструкция за употреба	23
NL Gebruiksaanwijzing	9	TR Kullanma talimatı	24
SV Bruksanvisning	10	EL Οδηγίες χρήσης	25
DA Brugsanvisning	11	RU Руководство по применению	27
NO Bruksanvisning	12	JA 取扱説明書.....	28
FI Käyttöohje.....	14	ZH 使用说明书.....	29
PL Instrukcja użytkowania.....	15	KO 사용 설명서.....	30
HU Használati utasítás	16		

INFORMATION

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INFORMATION

Date of last update: 2021-08-18

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

1.1 Function

Sealing sleeves improve the adhesion of lower limb prostheses and create an airtight seal on the prosthetic socket. The airtight seal makes it possible to produce a vacuum in the prosthetic socket.

Series (reference number)	Harmony (454A7, 454A8, 454A9)
----------------------------------	-------------------------------

The sealing sleeves are coated with a polyurethane gel on the inside. This gel stands out for its viscoelasticity. The sealing sleeves are intended for use in combination with outlet valves or vacuum pumps. The gaiter included in the scope of delivery protects the sealing sleeve against direct contact with the socket brim.

2 Intended use

2.1 Indications for use

The product is intended exclusively for exoprosthetic fittings after a transtibial amputation.

2.2 Lifetime

The product is a wear part, which means it is subject to normal wear and tear.

3 Safety

3.1 Explanation of warning symbols

WARNING Warning regarding possible serious risks of accident or injury.

CAUTION Warning regarding possible risks of accident or injury.

NOTICE

Warning regarding possible technical damage.

3.2 General safety instructions

WARNING!

Risk of severe injuries

- ▶ Store the product out of reach of children. There is a risk of suffocation if the product is pulled over the mouth and nose.

CAUTION!

Risk of injury and risk of product damage

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ Do not expose the product to temperatures above +80 °C or below -20 °C.
- ▶ Make sure the product is applied properly and fits correctly. Improper or excessively tight application can cause pressure points and constriction of blood vessels and nerves.
- ▶ Do not continue to use the product if it does not fit properly.

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may only be used by one person; reuse on another person is prohibited.
- ▶ Observe the cleaning instructions.
- ▶ Only use the product on intact skin.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Sealing sleeve	–
1	Gaiter	454A11

5 Preparing the product for use

5.1 Size selection

- 1) Determine the circumference at the height of the medial tibial plateau.
- 2) **If required for the sealing sleeve:** Determine the circumference 20 cm above the medial tibial plateau.
- 3) Select the sealing sleeve size that corresponds to the resulting measurement. The next smaller size can be chosen as an alternative.
- 4) Check the fit of the sealing sleeve.

5.2 Assembling

Gaiter

- 1) Pull the gaiter over the prosthetic socket with the Lycra side facing in. Position the lower edge of the gaiter about **2.5 cm** below the socket brim.
- 2) Secure the gaiter on the prosthetic socket with vinyl or polyethylene adhesive tape.

Sealing sleeve

- 1) Turn the gel side in the lower half of the sealing sleeve inside out.
- 2) Apply the sealing sleeve to the socket brim and unroll the half that was turned inside out over the prosthetic socket.

6 Use

CAUTION

Use of powder (e.g. baby powder or talcum)

Skin irritation on the residual limb and loss of function of prosthetic components due to clogging with particles or deprivation of lubricant

- ▶ Keep the product away from powders.

NOTICE

Mechanical overload

Impaired functionality due to mechanical damage

- ▶ Check the product for damage prior to each use.
- ▶ Do not use the product if its functionality has been impaired.

- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

6.1 Donning the prosthesis

> **Prerequisite:** The liner (and, if necessary, a residual limb sock) has been applied.

- 1) Roll the upper half of the sealing sleeve down to the prosthetic socket.
- 2) Roll the gaiter down over the socket brim.
- 3) Slide the residual limb into the prosthetic socket.
- 4) Roll the gaiter up, avoiding wrinkles wherever possible.
- 5) Roll the sealing sleeve up over the knee. Avoid wrinkles, air pockets and soft tissue displacement as you do so.

6.2 Cleaning and Care

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

- 1) Unroll the sealing sleeve from the prosthetic socket.
- 2) Hand wash the sealing sleeve in warm water (30 °C) using neutral detergent (e.g. 453H10=1 Ottobock Derma Clean).

NOTICE! Do not wring out the sealing sleeve, as this could damage it.

- 3) Rinse the sealing sleeve thoroughly with clear water until all detergent residues have been removed.
- 4) Put a hand towel into the sealing sleeve and leave the sealing sleeve to dry lying flat on a drying rack.

7 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

8.3 Warranty

The manufacturer warrants this device from the date of fitting. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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